

A black and white photograph of three women of diverse ethnicities smiling and looking towards the camera. The woman on the left is a white woman with long hair, the woman in the center is a Black woman with a large afro, and the woman on the right is an Asian woman with long dark hair. They are all wearing light-colored, thin-shoulder tops. The word "BONITTO" is overlaid in large white capital letters across the top of the image.

BONITTO

*A whole new look,  
a whole new world*

▪ **PROFESSIONAL SKIN CARE** ▪ **WIDE RANGE OF FILLERS**

[www.bonitto.co](http://www.bonitto.co)



# ALL IN-ONE SOLUTION

VARIOUS COMPOSITIONS  
OF MESOTHERAPIES

**WIDE RANGE OF FILLERS**

PROFESSIONAL SKIN CARE


**MAXIMUM SKIN TOLERANCE**

ISO 22716 & ISO 13485 & CE 0373 CERTIFIED



***BONITTO 01 PRO AGE FINE is an ultra-plastic gel of cross-linked hyaluronic acid intended for implantation in shallow wrinkle areas and moderate skin defects. The product is dedicated mainly to correcting the glabella, wrinkles around the mouth and eyes, neck, hands, or the tear through. The product ensures natural volumetric correction by increasing the volume of the tissue, which results in the tissue being accentuated to the desired result.***


# 01 PRO AGE FINE

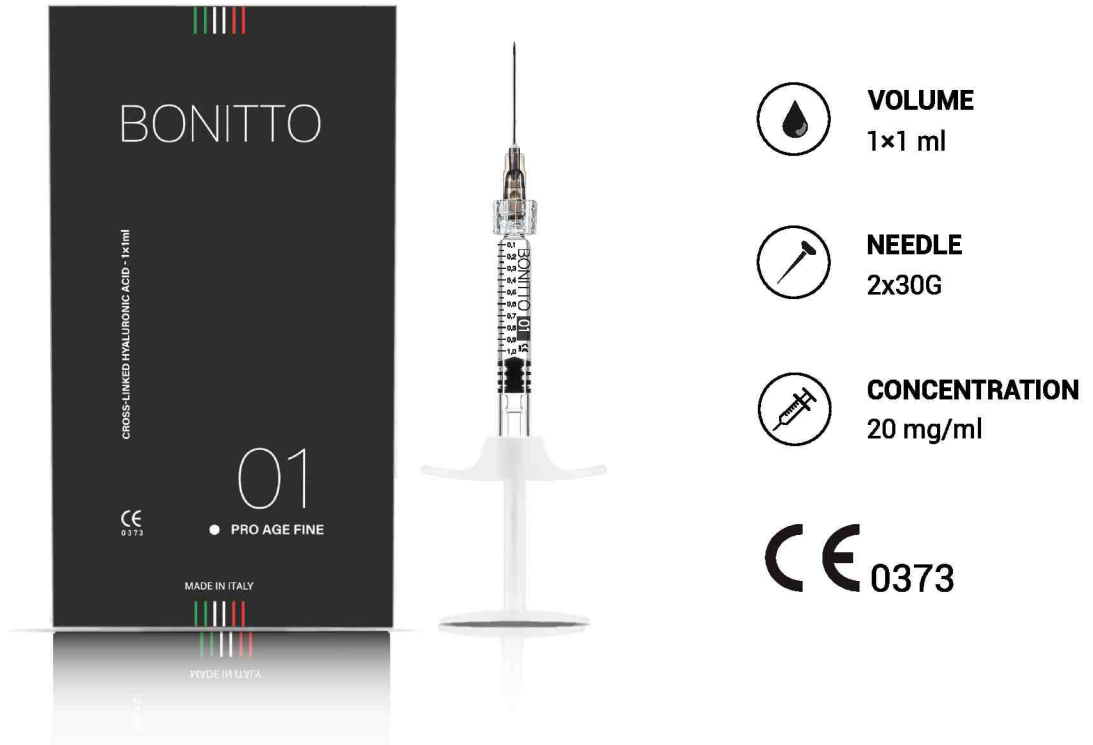

## RECOMMENDED INDICATIONS

- Smoothing fine wrinkles and shallow lines
- Rejuvenating the eye area: Minimizing wrinkles
- Lip revival: Delicate volume enhancement
- Enhancing themouth: Diminishing wrinkles
- Leveling wrinkles of the glabella
- Hand skin Bio-Complementation: Restore vitality
- Correcting neck wrinkles: Refining the appearance

# 02 LIPS PLUM

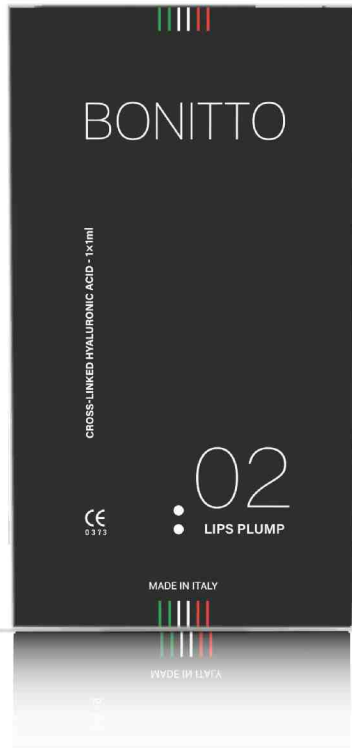

**VOLUME**  
1x1 ml



**NEEDLE**  
2x27G



**CONCENTRATION**  
21 mg/ml

**CE** 0373

## RECOMMENDED INDICATIONS

- Lip contouring and volume enhancement
- Modern techniques for flat lip modeling
- Lifting the corners of the mouth
- Correcting medium and deep wrinkles
- Filling nasolabial folds
- Smoothing puppet lines
- Hand skin Bio-Complementation



***BONITTO 02 LIPS PLUMP is an ultra-smooth gel of cross-linked hyaluronic acid intended for increasing soft tissue volume and correcting medium and deep wrinkles. The product is also dedicated for modeling contours and filling the volume of the lips and correcting soft tissue defects within the mouth such as: vertical wrinkles around the mouth, puppet lines and dropping corners.***



***BONITTO 03 AGE SOLUTION is an ultra-smooth gel of cross-linked hyaluronic acid dedicated for resting or increasing the volume of soft tissues and correcting deep wrinkles and kin grooves. The product is also dedicated to contour the shape of the face in areas such as: mandible line, bridge of nose, chin or nasolabial fold and puppet lines. The gel ensures visible volumetric correction by increasing the volume of the tissue, which results in the tissue being accentuated to the desired result.***



# 03 AGE SOLUTION



**VOLUME**  
1x1 ml

**NEEDLE**  
2x27G


**CONCENTRATION**  
24mg/ml

**CE** 0373

## RECOMMENDED INDICATIONS

- Correction of deep wrinkles and nasolabial folds
- Puppet lines
- Correction of the nose shape
- Contouring the mandible line
- Improving the chin shape
- Chin volumetry
- Correction of deep wrinkles

# 04 FACE SCULPT

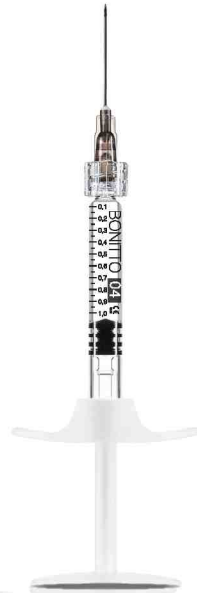

**VOLUME**  
1x1 ml



**NEEDLE**  
2x25G



**CONCENTRATION**  
26 mg/ml

**CE** 0373

## RECOMMENDED INDICATIONS

- Cheek volume enhancement
- Face modeling and contouring
- Nose shape correction
- Mandible line contouring
- Chin volumetry
- Very deep wrinkle correction



***BONITTO 04 FACE SCULPT is a strong gel of cross-linked, highly concentrated hyaluronic acid dedicated to increasing the volume of soft tissues and facial volume. The product is designed for contouring face shape in areas such as: mandible line, bridge of the nose, chin, or cheek volumetry.***



***BONITTO 05 VOLUME is a gel of cross-linked, highly concentrated hyaluronic acid dedicated for restoring or increasing the volume of soft tissues and modeling the body surface. It is perfect for leveling out extensive atrophies, inequalities created after liposuction treatments and volumetric body treatments. Designed for contouring the silhouette in areas such as buttocks, calves, or hands. This product provides a visible volumetric correction through an outstanding increase in tissue volume, which results in the enhancement of tissue to the desired point. Visibility and durability of the effect depend on the depth of application and the original condition of the skin***

# 05 VOLUME



**VOLUME**  
1x10 ml



**CONCENTRATION**  
20 mg/ml


**CE** 0373

## RECOMMENDED INDICATIONS

- Giving volume to flabby skin.
- Body shaping and contouring.
- Buttocks enlargement.
- Calf shape correction.

# TREATMENT WITH FILLERS

01	
01	
01	
04	03
	01
03	02
03	02
	03
	04
03	02
	04
	04

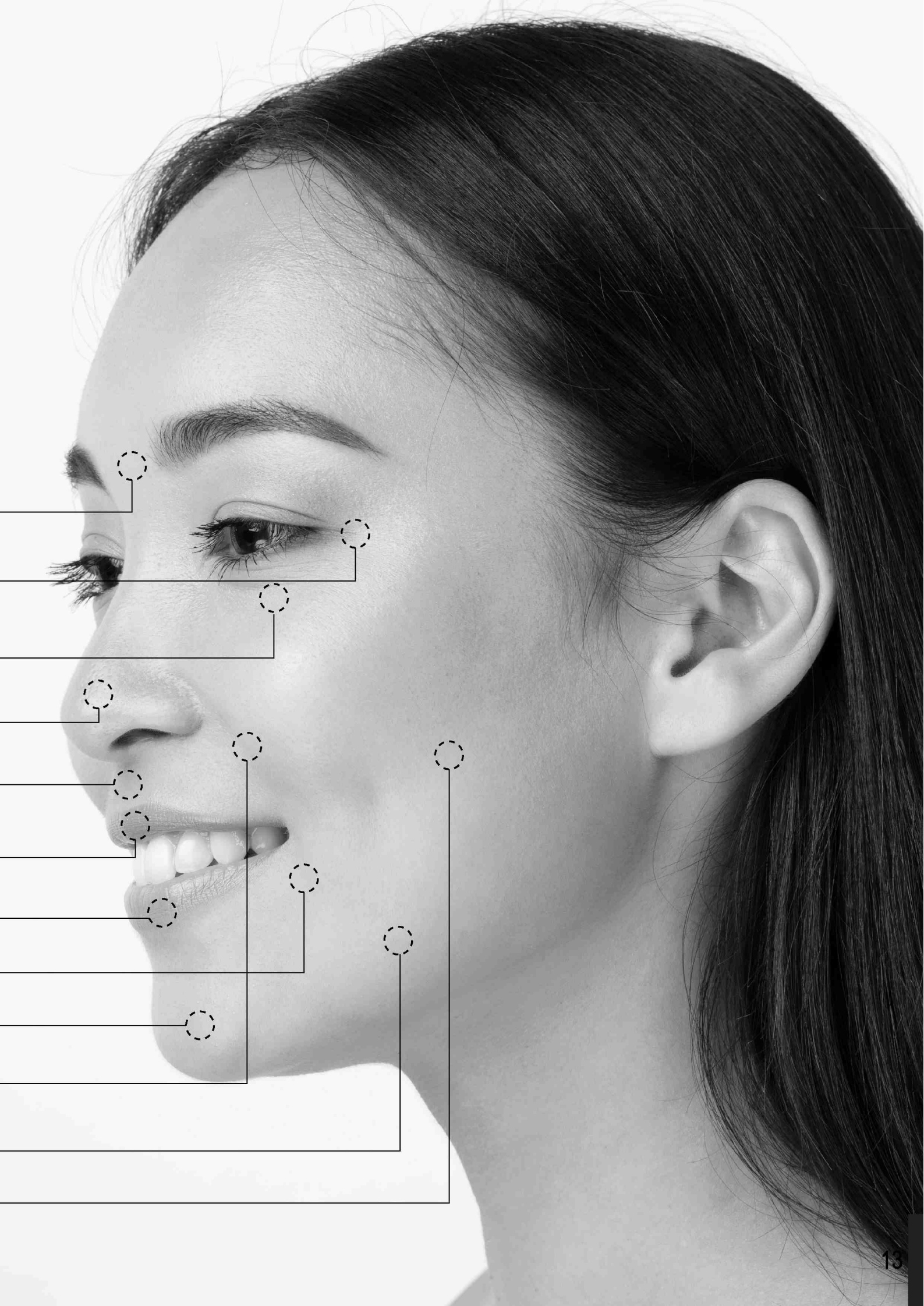



## HA+ BOOSTER & AA15

5x5ml

### ○ INDICATION

Bonitto HA+ BOOSTER & AA15 is the formula based on the beneficial properties of Hyaluronic Acid and Amino Acids that deeply moisturizes and strongly revitalizes the skin.

Hyaluronate Acid – a product with strong hygroscopic properties, making it one of the most effective ingredients to moisturize the skin and prevent excessive loss of water from its surface. It helps to maintain the stability of structural proteins – collagen and elastin, thus improving tension and elasticity, ensuring a youthful appearance.

The amino acid sequence stimulates the production and reconstruction of the skin's support fibers, causing strong restructuring and thickening of the tissue.

Perform one session each month for three months.

### ○ EFFECT

- Deep stimulation
- Intensive regeneration
- Elasticity of the skin
- Skin rejuvenation
- Long lasting moisturizing
- Firming effect
- Smooth wrinkles

### ○ ACTIVE INGREDIENTS

Hyaluronic Acid, Butylene Glycol, Thiamine HCL, Pyridoxine HCL, Niacinamide, Lysine, Histidine, Arginine, Aspartic Acid, Threonine Serine, Glutamic Acid, Proline, Alanine, Valine, Isoleucine, Leucine, Tyrosine, Phenylalanine.







## COLLAGEN ULTRA

5x5ml

### ○ INDICATION

Bonitto COLLAGEN ULTRA is a firming product for mature skin that requires revitalization and deep hydration. Immediate results in improving skin's natural brightness and moisture content.

The active ingredients contained in the product significantly improve the structure of the skin and provide deep hydration.

Hydrolyzed collagen is responsible for skin elasticity and firming, preventing the first signs of aging.

An advanced rejuvenating cocktail for a skin-revitalizing treatment. The unique properties which are enriched with ingredients stimulate long-term skin regeneration on various levels. This procedure restores the correct moisture balance, tension and elasticity of the skin and slows down the aging process of tissues.

Perform 3 session over 3 to 4 months.

### ○ EFFECT

- Strong skin nutrition
- Restores radiance
- Skin rejuvenation
- Slow down ageing
- Increase density of the dermis
- Improving skin color


### ○ ACTIVE INGREDIENTS

Hydrolyzed Collagen, Glycerin, Gleditsia Triacanthos Seed Extract, Sodium Hyaluronate.



## EY+

5x5ml

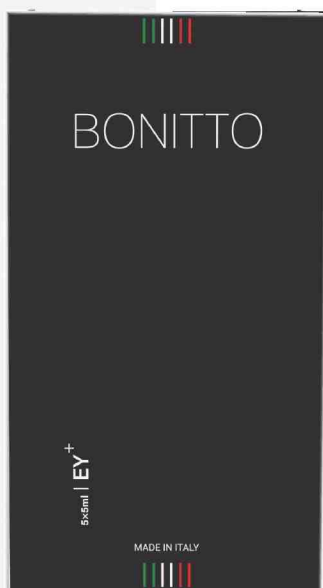

### ○ INDICATION

Bonitto EY + is revitalization complex for the eye area that requires exceptional care.

It is a sterile Sodium Hyaluronate -based booster which has a unique sequence. Supported by strong antioxidants: Niacinamide and Glutathione.

This mesotherapy is ideal for patients who have tissue atrophy. It will help to reduce the number of wrinkles. The result will remove dark circles under your eyes and reduce eyelid skin sagging. The rich drainage composition, the preparation works in many directions, allowing you to regain the perfect appearance of the eyes.

Perform a course of up to 3 treatments at regular intervals is recommended for best results.

### ○ EFFECT

Density and elasticity of the skin around eyes

Natural moisturizing

Preventing dryness


Guarantees hydration

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate ,Proline ,Serine ,Glycine ,Niacinamide ,Glutathione.

## AGE+

5x5ml

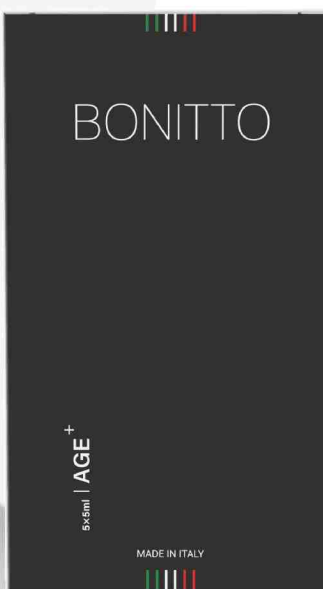

### ○ INDICATION

Bonitto AGE + is an extremely effective procedure for restoring healthy skin.

Accelerates natural regeneration. The active ingredients provide rapid stimulation of collagen, allow for quick extracellular matrix regeneration.

Powerful vitalization and nourishment of dehydrated skins and lifting of tired skin.

Perform session every three weeks (3-4 procedures).

### ○ EFFECT

Deep stimulation

Intensive regeneration

Rejuvenation

Revitalisation

Smooths wrinkles


### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Butylene glycol

**Amino Acids:** Lysine, Histidine, Arginine, Aspartic acid, Threonine, Serine, Glutamic acid, Proline, Alanine, Valine, Isoleucine, Leucine, Tyrosine, Phenylalanine.

## OX+

5x5ml

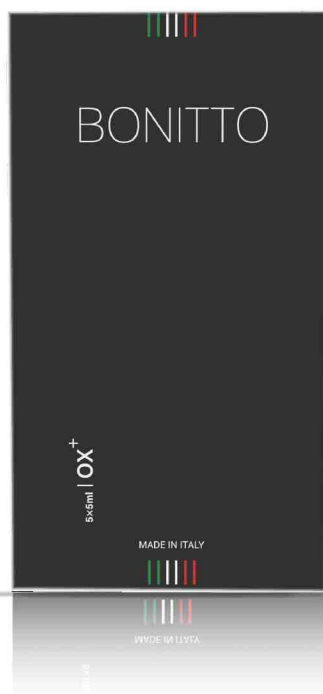

### ○ INDICATION

Bonitto OX+ is a sterile cosmetic solution containing a complex for skin hydration & stimulating.

Niacinamide, tranexamic acid, and glutathione contained in the product help minimize dermal discolorations and have strong brightening properties.

An innovative combination of succinic acid, hyaluronic acid, and Glycyrrhiza glabra Root Extract allows the skin to rehydrate, at the same time counteracting the activity of free radicals, all of which helps slow down aging processes.

Perform for session every 7-14 days of the procedure for 4/6 sessions.

### ○ EFFECT

Boosts natural renewal processes in the skin

Reducing the appearance of dark spots

Healthy skin tone


Skin stimulating

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Panthenol, NIACINAMIDE, SUCCINIC ACID, Proline, Glycine, Propylene Glycol, Glycyrrhiza glabra Root Extract, 3-O-ethyl ascorbic acid.

## SHINE POWER

5x5ml

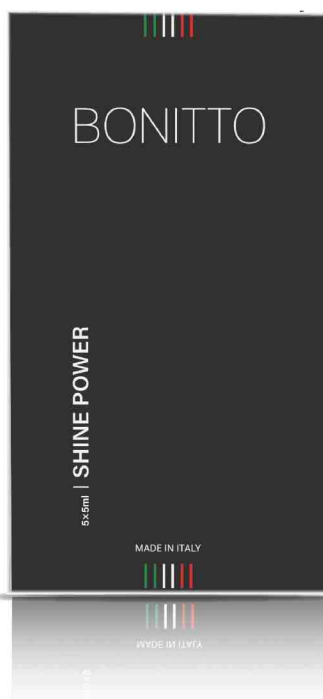

### ○ INDICATION

Bonitto SHINE POWER is a unique preparation for atopic type of the skin with the lighting and strengthening solution.

This cocktail features have a combination of antioxidants and active vitamin C, which has anti-inflammatory and stimulating effects.

This product can only be used by professionals, and by purchasing it, you are confirming that you are a trained medical or cosmetic professional.

Perform 1 ampoule every 7 days for 4/6 sessions.

### ○ EFFECT

Supports the stimulation of collagen

Protective skin

Improving

Elasticity

Less discoloration

### ○ ACTIVE INGREDIENTS

Ascorbic Glucoside, Succinic acid, Glutathione.

## HYAVITAL H+L

5x5ml

### ○ INDICATION

Bonitto HYAVITAL H+L is a mesotherapy that nourishes the skin, moisturizes the skin deeply, protects from daily attacks, and provides the nutrients it needs to function properly.

A rich complex of Amino Acids, specially designed for dull complexion, fine lines, large pores sizes, uneven pigmentation. This product provides the dermis with the essential ingredients it needs to thrive.

Sodium Hyaluronate smoothes out fine lines, restores radiance, deeply nourishes the skin, and increases the density of the dermis.

Perform a cycle of 6-8 sessions every two weeks.

### ○ EFFECT

Deep hydration

Elasticity

Improving skin color and quality

Strong nutrition and moisturisation

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate

**Amino Acids:** Lysine, Histidine, Arginine, Aspartic Acid, Threonine, Serine, Glutamic Acid, Proline, Glycine, Alanine, Valine, Isoleucine, Leucine, Tyrosine, Phenylalanine.

## HAIR ULTRA

5x5ml

### ○ INDICATION

Bonitto HAIR ULTRA is an advanced composition based on a combination peptide that extends the duration of a growth phase and improves the quality and life cycle of the hair.

This product is a modern technique aimed at improving the health of the scalp. Wide spectrum delivers the required amount directly to the hair follicles.

Perform from 1 to 2 times per week and the duration of the hair treatment is initially 3 months.

### ○ EFFECT

Revitalisation of hair and scalp

Reduction of excessive hair loss

Stimulation of hair growth


Improving blood circulation

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate

**Peptide Complex:**

sh-Polypeptide-11, sh-Polypeptide-1, sh-Oligopeptide-2, sh-Oligopeptide-10, sh-Polypeptide-9, sh-Oligopeptide-4  
OCTAPEPTIDE-2

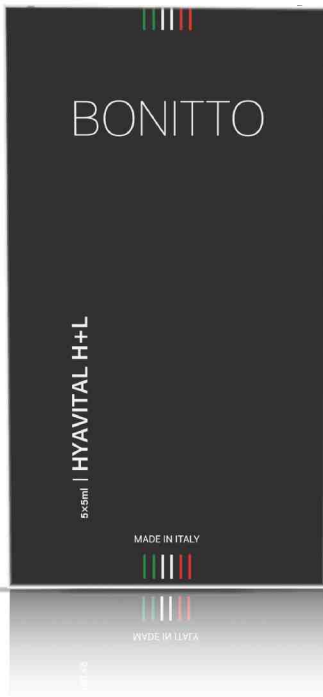



## OX ULTRA

5x5ml

### ○ INDICATION

Bonitto OX ULTRA is a sterile cosmetic solution containing regeneration and natural skin tone complex.

A specialized combination of succinic acid (1%) and non-crosslinked hyaluronic acid (0.55%) improves skin quality and structure, significantly increasing moisture levels and attacking lipids and proteins (including collagen and elastin), minimize the number of free radicals damage fibers and promote cell death.

Succinic acid lowering the risk of oxidative stress which causes irreversible changes in the DNA of the cell.

Sodium Hyaluronate increases tissue hydration, restoring an optimal environment for fibroblast activity. Slowing down processes that damage proteins. The combination of hyaluronic and succinic acids has demonstrated high efficiency in terms of skin regeneration and restoration of collagen in tissues. Two together act in synergy to counteract tissue hypoxia and degradation.

Perform for session every 7-14 days of the procedure for 4/6 sessions.

### ○ EFFECT

- Improves skin quality
- Increases the level of hydration
- Powerful anti-oxidative solution
- Increases tissue hydration
- Restoration of collagen in tissues
- Healthy and glowing skin

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Succinic acid, Acetyl decapeptide-3, Proline, Glycine, Hydroxyproline





## MEVITA15 +HA

5x5ml

### ○ INDICATION

Bonitto MEVITA15 +HA actively repairs the skin. Deficiency of the vital vitamins for the skin are combined in this advanced product.

This luxurious product has many uses including wrinkles and the back of hands, inside of legs and rms, as we as the abdomen and neck area. The rejuvenating cocktail is for skin revitalisation. Rich in ingredients stimulates multi-level skin regeneration in the long term.

Active ingredients will contribute to skin cell stimulation.

There are 15 Amino Acids that help produce elastin and collagen, two significant proteins making up the skin layers. Sodium Hyaluronate, it attracts moisture and reduces inflammation. When applied topically, sodium hyaluronate is excellent for reducing dryness and wrinkles. Perform every 3-5 months.

### ○ EFFECT

- Skin revitalization
- Age prevention
- Strong hydration
- Elimination of skin imperfections
- Skin nutrition
- Elimination of signs of aging

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Butylene glycol, Panthenol, Folic acid, Biotin, Inositol, Niacinamide, Pyridoxine HCL, Riboflavin, Thiamine HCL, Sodium Chloride, 3-O-ethyl ascorbic acid.

**Amino Acids:** Lysine, Histidine, Arginine, Aspartic Acid, Threonine, Serine, Glutamic Acid, Proline, Alanine, Valine, Isoleucine, Leucine, Tyrosine, Phenylalanine, Phenoxyethanol.





## LIPS ON

5x5ml

### ○ INDICATION

Bonitto LIPS ON is a revolutionary special formula to improve the vitality and color of your lips.

Concentrates rich in hyaluronic acid and nourishing substances rapidly increase moisture levels and smooth chapped lips.

The active compositions improve microcirculation, slightly increase the volume and pigmentation of the lips, making intense in color and shape. This product can stimulate the regeneration and renewal of the lips' protective barrier and restore perfect smoothness, softness and moisture.

Perform 4-6 sessions, at intervals of 14-21 days

### ○ EFFECT

Stimulating  
Smoothing  
Soothing  
Moisturizing  
Natural volume

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Sodium Ascorbate, Isohexadecane Ethylene/ Propylene/ Styrene Copolymer, Butylene/Ethylene/Styrene, Copolymer, Xanthan Gum, Phenoxyethanol, Tripeptide-1.

## LIPO+

5x10ml

### ○ INDICATION

Bonitto LIPO+ is a sterile cosmetic solution containing a complex for improvement of body contours and firmness.


Designed to eliminate excess fat tissue and shape the body. Leading to the effective reduction of local fat tissue and smoothing the body structure.

The content of the highest permissible concentration of Phosphatidylcholine, additionally enriched with peptides, guarantees quick and spectacular effects of reducing the tissue circumference, visible after the first procedure.

Perform once every 21 days for 4 sessions at most.

### ○ EFFECT

Reduction of fat  
Improvement of tissue tension and firmness  
Safe treatment  
Elimination of skin sagging

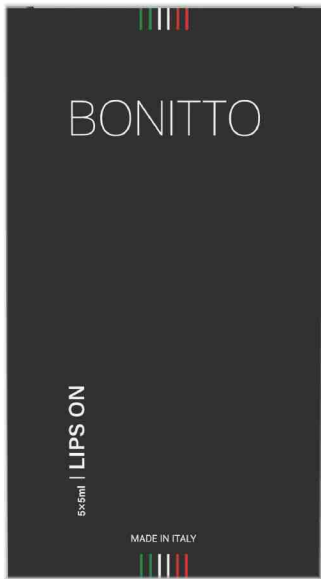

## PRO AGE

5x5ml

### ○ INDICATION

Bonitto PRO AGE is a vitamin cocktail that combats the signs of the first wrinkles.

It is essential in stimulating tissue regeneration, especially after prolonged exposure to harmful UV rays. Synthetic vitamin C can increase protection against free radicals, close blood vessels, and fade pigment spots. Active ingredients have remedial effects by improving skin blood supply, increases the synthesis of collagen and elastin.

Perform 1 ampoule every 7 days for 4/6 sessions.

### ○ EFFECT

Proper hydration  
Stimulating tissue renewal  
Elimination the first wrinkles  
Increases collagen

### ○ ACTIVE INGREDIENTS

Sodium Ascorbate, Tocopherol acetate, Niacinamide, Pyridoxine HCL, Riboflavin, Panthenol, Thiamine HCL.

## POWER OXY

5x5ml

### ○ INDICATION

Bonitto POWER OXY -the effective unique combination of active ingredients helps to rebuild skin processes and restore the protective barrier, fights against free radicals. The product is rich with active ingredients will provide antioxidant activity and protection of the skin. The skin will be refreshed, have a rested appearance, and begin to radiate health from the inside.


Perform 1 ampoule every 7 days for 4/6 sessions.

### ○ EFFECT

Discoloration solution  
Against free radicals  
Eliminates visible signs of aging  
Balanced and protective skin

### ○ ACTIVE INGREDIENTS

Retinyl Palmitate, Tocopherol Acetate, Ribes Nigrum (Black Currant), Bud Extract, Glycerine, Potassium Sorbate, Citric Acid, Sodium, Benzoate, Acetyl Cysteine, Taurine, Proline, Glutathione.

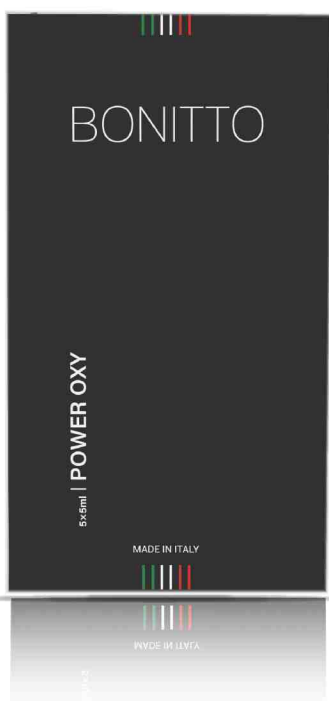

## BTX 2.0

5x5ml

### ○ INDICATION

Bonitto BTX 2.0 is a strong ageless complex for the skin with Sodium Hyaluronate and advanced compositions.

Acetyl octapeptide-3 - stable and effectively relieves facial wrinkles.

Pentapeptide-18 reduces the depth of wrinkles on the face caused by the contraction of muscles of facial expression, especially in the forehead and around the eyes.

Active ingredients designed to fight against mimic wrinkles and signs of skin ageing.

Perform a full cycle every six months.

### ○ EFFECT

Skin lifting

Preventing new wrinkles

Age prevention

Skin revitalization

Hydration and rejuvenation

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Succinic acid, Caprylyl Glycol Pentapeptide-18 , Acetyl Octapeptide- 3 , Proline, Glycine Hydroxyproline.

## DNA BOOSTER

5x5ml

### ○ INDICATION

Bonitto DNA BOOSTER is a solution recommended for skin that requires rejuvenation effect.

The product is based on sections of low-molecular DNA and RNA. Active ingredients of the product prepare tissues for an instant renewal and redermalization.

Perform 4 treatments at intervals of 10-14 days.

### ○ EFFECT

Collagen stimulation

Elastin stimulation

Skin structure improvement

Moisturising

Eliminate uneven skin tone

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Glutathione, Carnosine, Hydrolysed RNA, Hydrolysed DNA.





## PRO CELLS

5x10ml

### ○ INDICATION

Bonitto PRO CELLS is a unique solution of cellulite compression. An aesthetic technique that helps against cellulite and shape the contours of the figure.

The skin becomes smoother and more elastic, scars are smoothed out, and sagging areas are tightened.

With further completion of the course, the skin returns to a healthy, even tone, fat deposits are resolved, lymph outflow and blood circulation improve. This allows you to model the contours of the figure.


Perform 1 ampoule every 7 days for 4/6 sessions.

### ○ EFFECT

- Against loose skin
- Increases water retention
- Prevents swelling
- Stimulates lymph flow
- Eliminating cellulite

### ○ ACTIVE INGREDIENTS

Caffeine, Carnitine, Propylene Glycol, Cynara scolymus Leaf Extract, Equisetum arvense Extract, Ananas sativus Fruit Extract, Palmaris palmate extract.

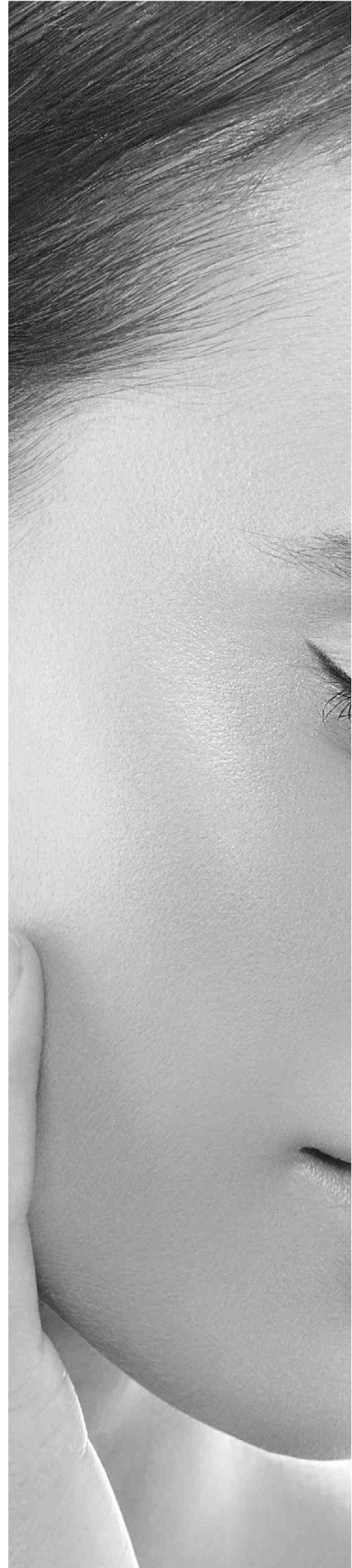



## WHITENING+

5x5ml

### ○ INDICATION

Bonitto WHITENING+ is a sterile cosmetic solution containing skin lightening complex.

Designed for skin depigmentation and whitening. Noticeably reduce pigment changes of the skin, improve skin tone, and restore brightness and vitality. The overall aim is to enhance your beauty and well-being with brighter and smoother skin.

Oligopeptide-34 (CG-TGP2) – Targeted peptide to reduce pigment.

Glutathione attracts attention in the arena of skincare, deeming it essential for healthy and nourished skin. It not only acts as an astonishing skin-brightening agent but is also known to reduce wrinkles, fine lines and improve skin elasticity.

### ○ EFFECT

Reduction of hyperpigmentation  
Elimination of free radicals  
Hydration & Revitalization of the skin  
Whitening

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, Panthenol, Glutathione, Oligopeptide-34 Tranexamic acid, Acetyl Glycyl  $\beta$ -alanine, Propylene Glycol, Glycyrrhiza, Glabra Root Extract.





## PEEL 4.0

5x10ml

### ○ INDICATION

Bonitto PEEL 4.0 is a cosmetic peeling containing alpha hydroxy acids for skin exfoliation.

### ○ APPLICATION SITE

Apply a thin layer of acid all over the face, neck and décolleté with the help of the appropriate dropper. Leave on for approximately 4/5 minutes. Carefully avoid the eye contour and mucous membranes, respect the application time and then rinse with cold water.

### ○ DIRECTIONS OF USE

It is possible to repeat a new session every 2-4 weeks for a maximum of 4 sessions. It is extremely important, during the weeks of treatment, to apply a regenerating moisturizing cream in the evening and an SPF 50+ sunscreen during the day, even after the treatment period, to protect the skin from the sun's rays.

### ○ DEGREE OF PENETRATION

The product is for external use only. Do not use the product if the container is damaged. Do not use the product during pregnancy and breastfeeding. Do not use the product in case of known incompatibility with any of the components of the preparation. Do not use the product after the expiry date shown on the packaging.

### SPECIFIC OR TARGET SPECIFIC PROBLEM

Enlarged pores and uneven skin structure, abnormal skin tone.

### ○ PRECAUTIONS

During all phases of handling the product, wear disposable latex and PE gloves. Apply exclusively to intact skin, avoiding the eye contour. If the product should penetrate the eyes, wash immediately and abundantly with running water, keeping the eyes wide open under the jet of water; remove any contact lenses.

### ○ PRODUCT STORAGE INSTRUCTION

If the person feels a strong burning sensation or notices intense redness, rinse immediately to remove the acid from the face. The application must be carried out by an experienced professional. The application is completed when the subject notices a slight redness and sensation of heat on the skin. Rinse thoroughly with cold water to remove the product. After application the skin may have redness and flaking on the treated area. The product must be stored in a cool place and must not be exposed to sunlight or heat sources. Close the container tightly after use.

### ○ EFFECT

High effectiveness and immediate effect  
Skin stimulation  
Painless and safe  
Whitening and depigmentation



## RADIANCE PRO

5x5ml

### ○ INDICATION

Bonitto RADIANCE PRO is concentrated transformation and care of the hands.

The sensitive skin of the hands, which is constantly exposed to external influences, requires sufficient moisturizing and nourishing ingredients with the ability to activate tissue regeneration functions.

Active ingredients are responsible for skin tension and density. Vitamin B3 forms a protective barrier that reduces water loss in tissues and inhibits the secretion of free radicals.

The correct composition of amino acids ensures proper fat metabolism and antioxidant activity, preventing aging of sensitive hand skin.

Perform 1 ampoule every 7 days for 4/6 sessions.

### ○ EFFECT

Strong hygroscopic effect

The right level of skin hydration

Protects the skin against UV radiation

Sufficient moisturizing and nourishing

### ○ ACTIVE INGREDIENTS

Sodium Hyaluronate, NIACINAMIDE, Acetyl Cysteine, Arginine HCL Proline, Glycine, Acetyl decapeptide-3.







## **BTX+**

5x5ml

### **○ INDICATION**

Bonitto BTX + is a sterile cosmetic solution containing a complex helpful in preventing age signs. Effect like Botox, a new generation designed to support the rebuilding of collagen fibers, increasing skin density and tension.

“The Botox-like” effect is based on muscle relaxation and strong hydration of the skin, the face regains its youthful contours and visibly smoothes out.

The right composition replenishes the deficiency of naturally occurring Sodium Hyaluronate, filling wrinkles from the inside and reducing those caused by dehydration.

Acetyl Hexapeptide-8 is a skincare ingredient that visibly reduces the appearance of fine lines, and wrinkles, & boosts collagen production. It is the first peptide that targets areas prone to developing expression lines.

Perform one session every 10-14 days.

### **○ EFFECT**

- Skin lifting
- Prevention of new wrinkles
- Age prevention
- Strong vitalization
- Nutrition of dehydrated skin
- Skin revitalization & rejuvenation
- Powerful antioxidant

### **○ ACTIVE INGREDIENTS**

Sodium Hyaluronate, Acetyl Hexapeptide-8, DMAE.

# CERTIFICATES

Organismo Notificatore 9373  
Notified Body 9373

**Istituto Superiore di Sanità**

Certificato n° QCT-0149-20 Addendum n° 1/11

**DICHIARAZIONE CE DI CONFORMITÀ**  
**SISTEMA COMPLETO DI GARANZIA DI QUALITÀ**

**DECLARATION of CONFORMITY**  
**FULL QUALITY ASSURANCE SYSTEM**

Il Titolare Superiore di Sanità, Organismo Notificatore 9373, certifica che il Sistema completo di garanzia della qualità soddisfa:

The Istituto Superiore di Sanità, Notified Body 9373, certifies that the total quality assurance system complies with:

VIDA S.p.A.  
Sede Legale/Registered Office: Via G. Galilei 4 - 56040 Crespinina - Lavoria (PI) ITALIA

per il dispositivo/for the device

Il Direttore dell'Organismo Notificatore  
The Director of Notified Body  
(Dichiarazione Responsabile)  
(Responsible Declaration)  
Roberto Torricelli

Organismo Notificatore 9373  
Notified Body 9373

**Istituto Superiore di Sanità**

ALLEGATO TECNICO  
TECHNICAL SHEET

Il Certificato n° QCT-0149-20 Addendum n° 1/11

di cui il presente allegato tecnico è parte integrante, è da considerarsi riferito solo alle apparecchiature prodotte sottoposte a sorveglianza.

of which this technical sheet is an integral part, refers only to the following products that are subject to surveillance.

Classe III (Class III)
Nome prodotto (Product name)
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile

Valutazione della conformità: MOD 341-01-01 n° 9320  
Conformity assessment: MOD 341-01-01 n° 9320

Il Direttore dell'Organismo Notificatore  
The Director of Notified Body  
(Dichiarazione Responsabile)  
(Responsible Declaration)  
Roberto Torricelli

**Bureau Veritas Certification**

**VIDA SRL**

Via Galileo Galilei, 4 - 56040 CRESPINA LORENZIANA (PI) - Italy  
Certified site:  
Via Galileo Galilei, 4 - 56040 CRESPINA LORENZIANA (PI) - Italy

Bureau Veritas Italia S.p.A. certifies that the Organization above and the cosmetic product produced, have been audited and found to be in accordance with the requirements of:

"Particular regulation for ISO 22718 certification" (Ann. 03) in accordance with the **UNI EN ISO 22718**

"Cosmetics, Good Manufacturing Practices (GMP), Guidelines on good manufacturing practices"

Scope of certification:  
Production, packaging and storage of cosmetic solutions and pastes.

Original cycle start date: 11-May-2019  
Expiry date of previous cycle: 10-April-2021  
Certification / Recertification Audit date: 10-April-2021  
Certification / Recertification cycle start date: 04-May-2021  
Subject to the continued satisfactory operation of the organization's Management System, this certificate expires on: 10-May-2024

Certificate No.: IT338182 Version: 1 Issue Date: 29-January-2023

**ACCREDIA**

GIORGIO LANZAFAME - Local Technical Manager

Certification body address:  
Bureau Veritas Italia S.p.A., Viale Monza, 347 - 20139 Milano, Italia

**ESAME DI CERTIFICAZIONE**

Il presente documento è un estratto del rapporto di certificazione. Per informazioni e per il documento completo, si prega di rivolgersi al Centro di Assistenza Clienti (CCC) al numero verde 800 20 20 20.

Il Cliente è tenuto a mantenere in vigore il presente documento e a non utilizzarlo per scopi diversi da quelli per i quali è stato emesso.

Il Cliente è tenuto a mantenere in vigore il presente documento e a non utilizzarlo per scopi diversi da quelli per i quali è stato emesso.

Il Cliente è tenuto a mantenere in vigore il presente documento e a non utilizzarlo per scopi diversi da quelli per i quali è stato emesso.

Il Cliente è tenuto a mantenere in vigore il presente documento e a non utilizzarlo per scopi diversi da quelli per i quali è stato emesso.

**Vida**

VIDA SRL, as a manufacturer of medical devices such as Opera fillers and BONITTO fillers, declares that Opera fillers and BONITTO fillers are equivalent devices and differ only in their trade name. The documentation referring to the Opera devices therefore also applies to the BONITTO devices. The BONITTO device is marketed in the country-of-Origin Italy with CE number: QCT-0149-20, EPG-0289-20. The equivalent medical device Opera is marketed in the country-of-Origin Italy with the same CE number: QCT-0149-20, EPG-0289-20.

BONITTO	Correspondence with Opera product name
BONITTO 01	Opera I
BONITTO 02	Opera II
BONITTO 03	Opera III
BONITTO 04	Opera IV
BONITTO 05	Opera V

Date: 02/02/24

CED:

**Bureau Veritas Certification**

**VIDA SRL**

Via Galileo Galilei, 4 - 56040 CRESPINA LORENZIANA (PI) - Italy  
Certified site:  
Via Galileo Galilei, 4 - 56040 CRESPINA LORENZIANA (PI) - Italy

Bureau Veritas Italia S.p.A. certifies that the Management System of the above organization has been audited and found to be in accordance with the requirements of the management system standards specified below:

**EN ISO 13485:2016**

Scope of certification:  
Design and manufacturing of medical devices for topical and injectable use, marketing of non-active medical devices for aesthetic medicine.

Original Cycle Start Date: 14-December-2017  
Expiry date of previous cycle: 13-December-2022  
Certification / Recertification Audit date: 23-November-2023  
Certification / Recertification Cycle Start date: 11-December-2023  
Subject to the continued satisfactory operation of the organization's Management System, this certificate expires on: 13-December-2026

Certificate No.: IT337802 Version: 1 Revision date: 11-December-2023

**ACCREDIA**

GIORGIO LANZAFAME - Local Technical Manager

Certification body address:  
Bureau Veritas Italia S.p.A., Viale Monza, 347 - 20139 Milano, Italy

Organismo Notificatore 9373  
Notified Body 9373

**Istituto Superiore di Sanità**

Certificato n° EPG-0289-20 Addendum n° 1/11

**ESAME CE DELLA PROGETTAZIONE DEL PRODOTTO**

**CE DESIGN-EXAMINATION CERTIFICATE**

Il Titolare Superiore di Sanità, Organismo Notificatore 9373, certifica che il Sistema di progettazione del dispositivo medico:

The Istituto Superiore di Sanità, Notified Body 9373, certifies that the design device relating to the medical device:

È stato sottoposto a verifica, conformemente ai requisiti dell'Allegato II (4), della Direttiva 93/42/CEE, e successive modifiche ed integrazioni.

has been submitted to verification, in accordance to Annex II (4), of Council Directive 93/42/EEC and subsequent modifications and integrations.

Il Direttore dell'Organismo Notificatore  
The Director of Notified Body  
(Dichiarazione Responsabile)  
(Responsible Declaration)  
Roberto Torricelli

Organismo Notificatore 9373  
Notified Body 9373

**Istituto Superiore di Sanità**

ALLEGATO TECNICO  
TECHNICAL SHEET

Il Certificato n° EPG-0289-20 Addendum n° 1/11

di cui il presente allegato tecnico è parte integrante, è da considerarsi riferito solo alle apparecchiature prodotte sottoposte a sorveglianza.

of which this technical sheet is an integral part, refers only to the following products that are subject to surveillance.

Classe III (Class III)
Nome prodotto (Product name)
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile
Opera <sup>®</sup> sterile

Valutazione della Conformità: MOD 341-20-01 n° 9320  
Conformity assessment: MOD 341-20-01 n° 9320

Il Direttore dell'Organismo Notificatore  
The Director of Notified Body  
(Dichiarazione Responsabile)  
(Responsible Declaration)  
Roberto Torricelli





[www.bonitto.co](http://www.bonitto.co)